

# Endwarts Freeze



**Category**  
Skin Care and Antiseptics  
**Manufacturer**  
MYLAN  
**Pack size**  
7.5g  
**RRP**  
£15.99  
**Legal Status**  
MD



## Uses

Device based on cryotherapy method for the removal of warts and verrucas from hands and feet.



## Format and Ingredients

Freezing agent containing:

N20 gas,



## Dosage

Turn the white part one full turn in the direction of the arrow until the star is back in the shaded zone. Push down the white part for exactly 2 seconds Place upright on a table with the transparent part down. Hold the device with a steady grip and with the other hand, push the white part downward for exactly 2 seconds. Remove the cap. If there is no ice vapour, repeat step 2. Immediately press the tip firmly against the wart for 15 seconds on hands, fingers and arms and for 40 seconds on feet.